Invacare® Cadiz

H291 / H296

en	Shower stool / Shower chair User Manual	3
de	Duschhocker/Duschstuhl Gebrauchsanweisung	5
fr	Tabouret de douche/Chaise de douche Manuel d'utilisation	8
it	Sgabello da doccia / Sedia da doccia Manuale d'uso	11
es	Taburete de ducha / Silla de ducha Manual del usuario	14
pt	Banco de duche/cadeira de duche Manual de utilização	16
nl	Douchekrak / douchestoel Gebruiksaanwijzing	19
no	Dusjkrakk/dusjstol Bruksanvisning	22
cs	Sprchový stoleček / sprchová židle Návod k obsluze	25
hr	Stolac za tuširanje bez naslona / stolac za tuširanje s naslonom Upute za uporabu	27
hu	Zuhanyzószármoly / Zuhanyzószék Használati utasítás	30
sk	Sprchovací stolček/stolička Návod na obsluhu	33
lv	Dušas taburete/dušas krēsls Instrukciju rokasgrāmata	36

INVACARE

XXXXXXXXX (K) (B) (C)

REF (J) XXXXXXXX

MD (L) G F E

YYYY-MM XXX kg

LOT (D)

UDI (H) (01) 04028698123456(10) 00AAA1234

4 10 2869 811 23456

UK CA CE

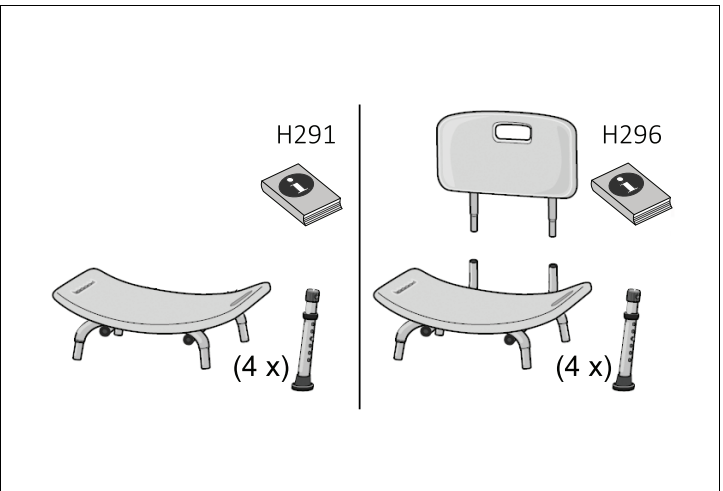
Fig. 1

Fig. 2

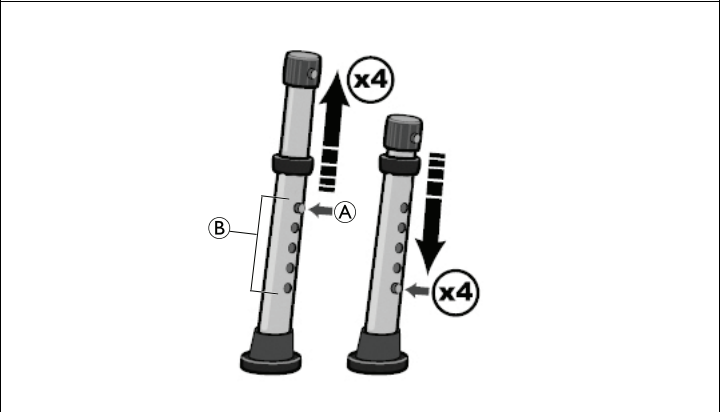
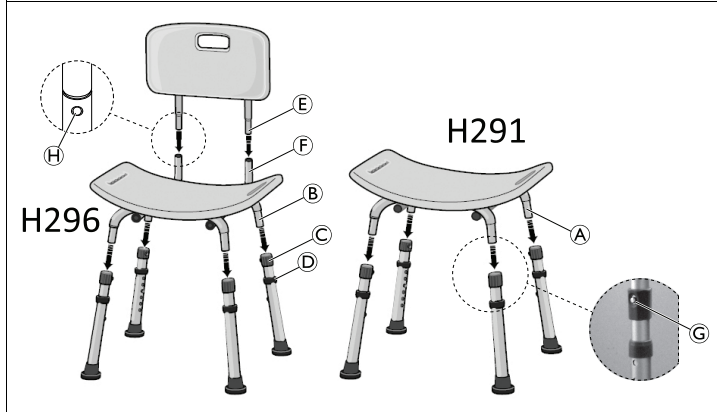
Fig. 3

Fig. 4

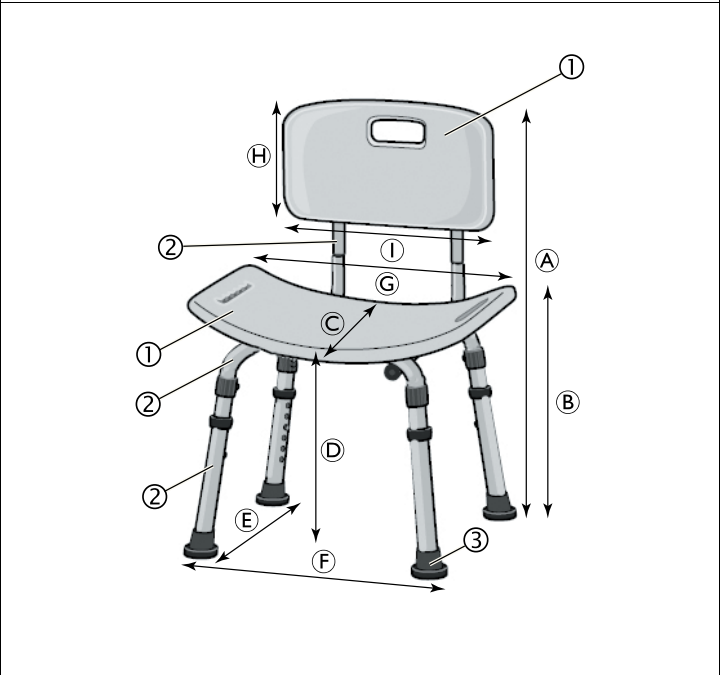
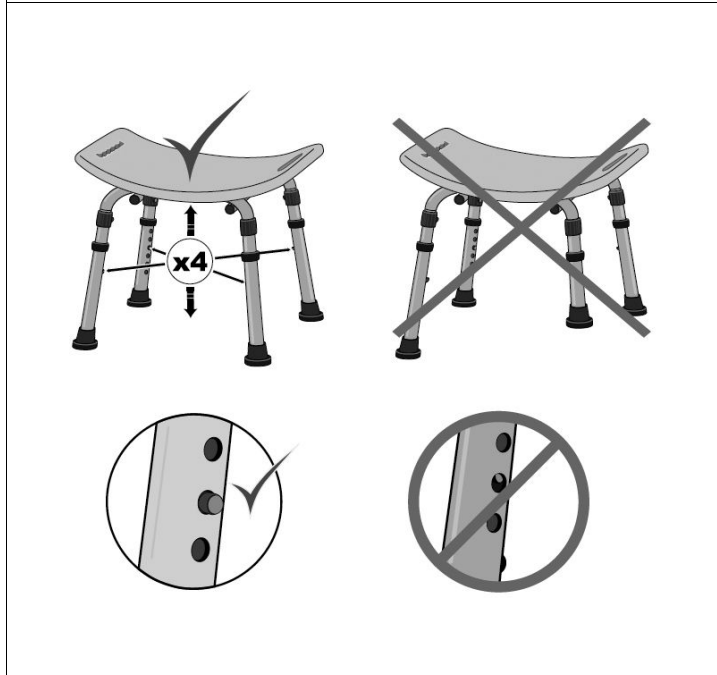
Fig. 5

Fig. 6

1 General

1.1 Introduction

This user manual contains important information about the handling of the product. To ensure safety when using the product, read the user manual carefully and follow the safety instructions.

Invacare reserves the right to alter product specifications without further notice.

Before reading this document, make sure you have the latest version. You find the latest version as a PDF on the Invacare website.

If you find that the font size in the printed document is difficult to read, you can download the PDF version from the website. The PDF can then be scaled on screen to a font size that is more comfortable for you.

In case of a serious incident with the product, you should inform the manufacturer and the competent authority in your country.

1.2 Warranty Information

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business in the respective countries.

Warranty claims can only be made through the provider from whom the product was obtained.

1.3 Compliance

Quality is fundamental to the company's operation, working within the disciplines of ISO 13485.

This product features the CE mark, in compliance with the Medical Device Regulation 2017/745 Class I.

This product features the UKCA mark, in compliance with Part II UK MDR 2002 (as amended) Class I.

We are continuously working towards ensuring that the company's impact on the environment, locally and globally, is reduced to a minimum.

We only use REACH compliant materials and components.

Product-specific Standards

The product has been tested and conforms to the standard ISO 17966 (Assistive products for personal hygiene that support users) and related standards. For further information about local standards and regulations, contact your local Invacare representative. See addresses at the end of this document.

1.4 Service Life

The expected service life of this product is three years when used daily and in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

1.5 Limitation of Liability

Invacare accepts no liability for damage arising from:

- Non-compliance with the user manual
- Incorrect use
- Natural wear and tear
- Incorrect assembly or set-up by the purchaser or a third party
- Technical modifications

- Unauthorised modifications and/or use of unsuitable spare parts

2 Safety

2.1 Safety Information

WARNING!

Risk of Serious Injury or Damage

Improper use of this product may cause injury or damage.

- If you are unable to understand the warnings, cautions or instructions, contact a health care professional or provider before attempting to use this equipment.
- Do not use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manual, service manual or instruction sheet supplied with this product or optional equipment.

WARNING!

Risk of falling!

- Do not use if defective.
- Do not make any unauthorized alterations or modifications to the product.
- Do not move or displace the product under load or during use.
- Do not stand on the product.
- Do not use or store the product outdoors.
- Do not tilt.
- Do not sit on the product if not all four legs have full contact to the level ground.
- Do not use for transfer from the side or partial load.

WARNING!

- Before first use, clean the product thoroughly.
- Before every use, check the product for damage and ensure that it is securely assembled.

CAUTION!

Risk of Injury

Non-original or wrong parts may affect the function and safety of the product.

- Only use original parts for the product in use.
- Due to regional differences, refer to your local Invacare catalogue or website for available options or contact your Invacare distributor. See addresses at the end of this document.

2.2 Labels and Symbols on the Product (Fig. 1)

(A)	Manufacturer	(G)	Production date
(B)	UK Conformity Assessed	(H)	Unique Device Identifier
(C)	European Conformity	(I)	Medical Device
(D)	Lot number	(J)	Reference number
(E)	Read user manual	(K)	Product designation
(F)	Maximum user weight		

The identification label is attached to the underside of the seat.

3 Product Overview

3.1 Intended Use

The Invacare Cadiz is a shower stool / shower chair composed of four anti-slip height adjustable feet and a backrest (if ordered). This product is intended to be used as a shower aid.

Intended User

Users are any adults and adolescents with restricted or limited mobility or limited muscle strength. If used without an assistant, the users must be able to keep their own balance.

Indication

- Restricted or limited mobility (e.g.: legs and/or hips) or
- having balance issues to stand up or walk due to transitional or permanent disabilities.

There are no contraindications known when using the product as intended. For the maximum user weight, see 7 *Technical Data, page 5*.

3.2 Scope of Delivery

The scope of delivery depends on the version ordered. The possible versions are shown in figure 2.

4 Setup

CAUTION!

Risk of Finger Trapping

- Take care during assembly.

Installing the Legs (Fig. 3)

1. Place the seat upside down on a clean, flat and stable surface (floor or table).
2. Completely insert the four legs into the tube ends **A** (for H291) respectively **B** (for H296) on the underside of the seat.

WARNING!

Risk of Falling

- Make sure that all four snap buttons **C** engage in the leg tube holes with an audible click.

Installing the Backrest (H296)

1. Place the shower chair on its legs.
2. Completely insert both tubes **E** of the backrest into the tubes **F** of the seat.

WARNING!

Risk of Falling

- Make sure that both snap buttons **H** of the backrest tubes fully engage in the holes of the seat tubes with an audible click.

Adjusting the Seat Height (Fig. 4)

The height of the seat can be adjusted between 405 and 505 mm in five steps of 25 mm.

1. Place the seat upside down on a clean, flat and stable surface (floor or table).
2. Press the snap button **A** of the legs and slide the leg extension up or down to the desired adjustment hole **B**.

WARNING!

Risk of Falling

- Make sure that all four legs are adjusted to the same height, see Fig 5.
- Make sure that all four snap buttons engage in the leg holes with an audible click, see Fig 5.

5 Maintenance

5.1 General Maintenance Information

This product is maintenance-free, provided that the cleaning and disinfection instructions are adhered to.

5.2 Cleaning and Disinfection

General Safety Information

CAUTION!

Risk of Contamination

- Take precautions for yourself and use appropriate protective equipment.

NOTICE!

Wrong fluids or methods can harm or damage the product.

- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean.
- Never use corrosive fluids (alkalines, acid etc.) or abrasive cleaning agents. We recommend an ordinary household cleaning agent such as dishwashing liquid, if not specified otherwise in the cleaning instructions.
- Never use a solvent (cellulose thinner, acetone etc.) that changes the structure of the plastic or dissolves the attached labels.
- Always make sure that the product is completely dried before taking it into use again.

For cleaning and disinfection in clinical or long-term care environments, follow your in-house procedures.

Cleaning Intervals

NOTICE!

Regular cleaning and disinfection enhances smooth operation, increases the service life and prevents contamination.

Clean and disinfect the product:

- regularly while in use,
- before and after any service procedure,
- when it has been in contact with any body fluids,
- before using it for a new user.

Cleaning

NOTICE!

Maximum cleaning temperature:
– 60 °C by hand

1. Remove detachable parts.
2. Clean the product by wiping down all generally accessible surfaces.
3. Dry the parts with a cloth.

Disinfection Instructions

Method: Follow the application notes for the used disinfectant and wipe-disinfect all accessible surfaces.

Disinfectant: Ordinary household disinfectant.

Drying: Allow the product to air-dry.

6 After Use

6.1 Reconditioning

This product is suitable for reuse. To recondition the product for a new user, carry out the following actions:

- Inspection
- Cleaning and disinfection

For detailed information, see 5 *Maintenance*, page 4 .

Make sure that the user manual is handed over with the product.

If any damage or malfunction is detected, do not reuse the product.

6.2 Disposal

Be environmentally responsible and recycle this product through your recycling facility at its end of life.

Disassemble the product and its components, so the different materials can be separated and recycled individually.

The disposal and recycling of used products and packaging must comply with the laws and regulations for waste handling in each country. Contact your local waste management company for information.

7 Technical Data

7.1 Dimensions, Weights and Materials (Fig. 6)

	H291	H296
Ⓐ	–	775 – 875 mm
Ⓑ	450 – 550 mm	
Ⓒ	310 mm	
Ⓓ	405 – 505 mm	
Ⓔ	395 – 405 mm	
Ⓕ	465 – 480 mm	
Ⓖ	510 mm	
Ⓗ	–	230 mm
Ⓘ	–	400 mm

Weights	
Weight H291	2.4 kg
Weight H296	3.0 kg
Maximum user weight	143 kg

①	PE-HD	③	TPE
②	Aluminum		

All product components are corrosion resistant and do not contain natural rubber latex.

7.2 Environmental Parameters

Operating Conditions	
Ambient temperature	10 - 40 °C
Relative humidity	15% - 93%, non-condensing
Storage and Shipping Conditions	

Ambient temperature	0 - 40 °C
Relative humidity	30% - 75%, non-condensing

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Invacare Poirier SAS
Invacare Austria GmbH
Invacare GmbH
Invacare UK Operations Limited