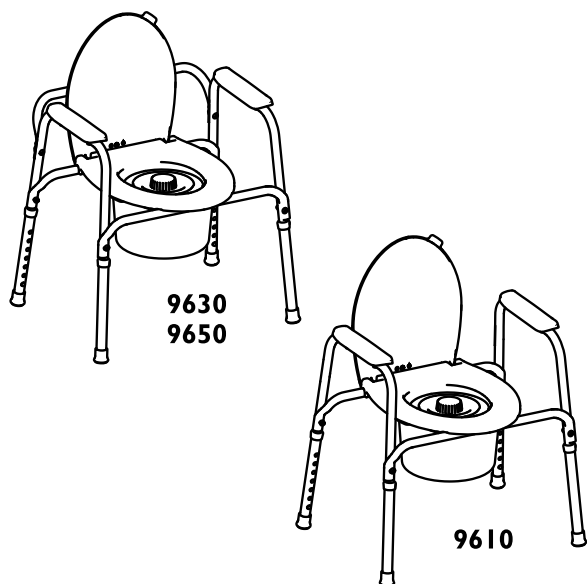
Invacare® Commodes

9610, 9630, 9650

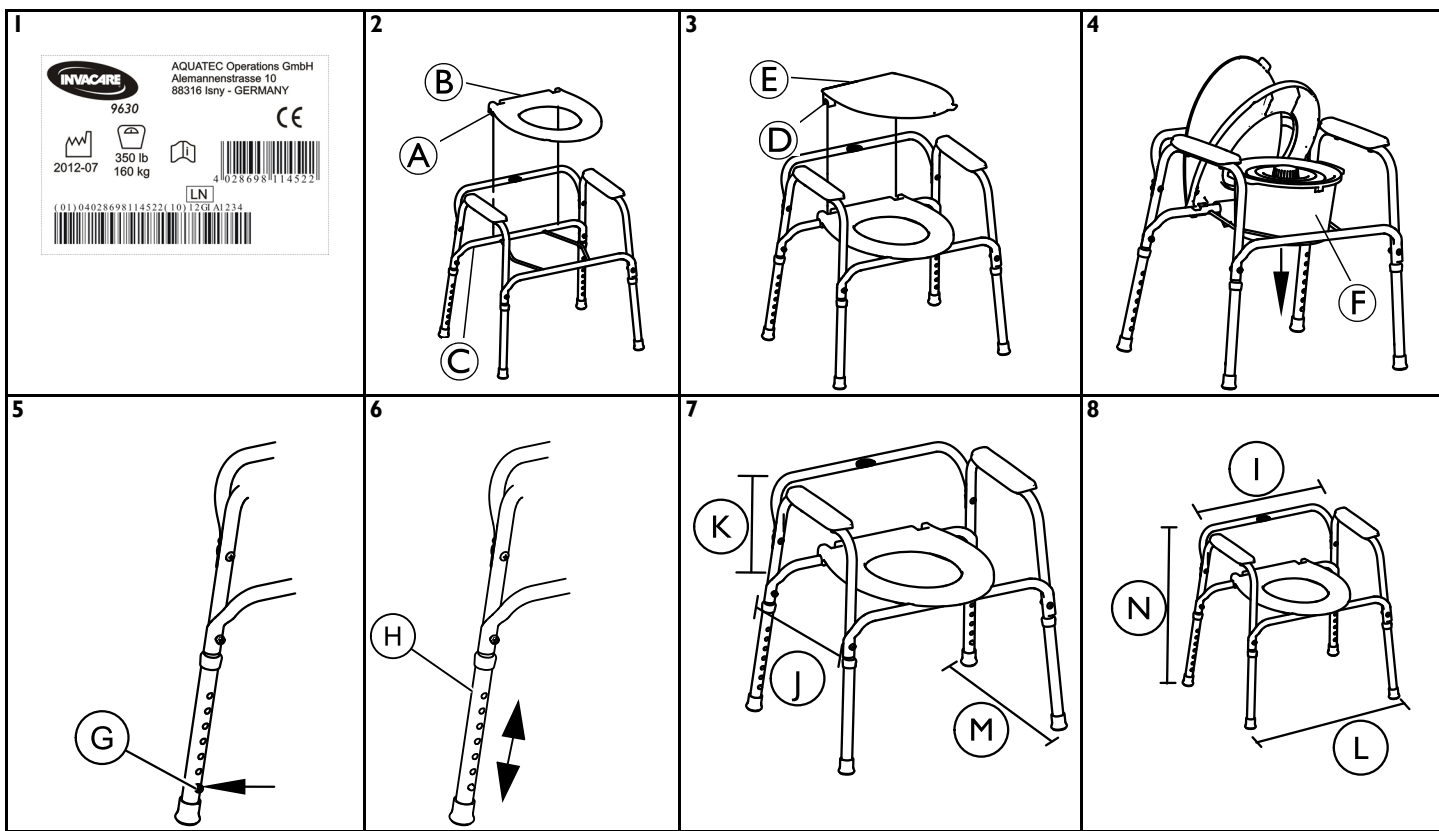
EN	Standard Commode	
	User Manual	1
DE	Standard-Toilettenstützgestell	
	Gebrauchsanweisung	4
ES	Inodoro estándar	
	Manual del usuario	6
FR	Chaise percée standard	
	Manuel d'utilisation	8
IT	Comoda standard	
	Manuale d'uso	10
NL	Standaardtoiletstoel	
	Gebruiksaanwijzing	12
PT	Cadeira sanitária 3 em 1 standard	
	Manual de utilização	15
SV	Standardhygienstol	
	Bruksanvisning	17
NN	Standard toalettstol	
	Bruksanvisning	19

EN

I General

I.1 Figures

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This manual **MUST** be given to the user of the product.
BEFORE using this product, read this manual and save for future reference.

Yes, you can.®

1.2 Symbols

Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. See the information below for definitions of the signal words.

WARNING! Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

CAUTION! Caution indicates a potentially hazardous situation which, if not avoided, may result in property damage or minor injury or both.

Gives useful tips, recommendations and information for efficient, trouble-free use.

IMPORTANT indicates a hazardous situation that could result in damage to property if it is not avoided.

This product complies with Directive 93/42/EEC concerning medical devices.

The launch date of this product is stated in the EC declaration of conformity.

1.3 Intended Use

The commode chair is to be used solely as an aid for going to the toilet by people who have:

- Difficulty standing or keeping their balance
- Weak legs or lower limb disability
- Medical conditions that impair their balance

The use of the toilet support frame is indicated with:

- Patients with a lack of trunk control, lack of lateral support
- Individuals with balance disorders
- People with serious malfunctioning of the lower extremities
- Individuals who are significantly impaired or have a disability of the lumbar spine, hip, or knee.

Any other use is prohibited.

Temperature for use: 10–40° C (50–104° F)

Max. user weight: 160 kg (350 lb) (25 stone).

1.4 Service Life

The expected service life of this product is five years when used in accordance with the safety instructions, maintenance intervals and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

Maintenance should be done at least every six months.

2 Technical Data

2.1 Dimensions and Weight

Refer to 1.1 Figures , page 1 for Dimensions.

Product	9610	9630	9650
①	14.17 in (360 mm)	14.17 in (360 mm)	14.17 in (360 mm)
②	16.54 in (420 mm)	16.54 in (420 mm)	16.54 in (420 mm)
Ⓚ	16.14–22 in (410–560 mm), in 7 steps	16.14–22 in (410–560 mm), in 7 steps	16.14–22 in (410–560 mm), in 7 steps
Ⓛ	24.4 in (620 mm)	24.4 in (620 mm)	24.4 in (620 mm)
Ⓜ	19.3 in (490 mm)	19.3 in (490 mm)	19.3 in (490 mm)

Product	9610	9630	9650
Ⓝ	23.62–29.53 in (600–750 mm)	23.62–29.53 in (600–750 mm)	23.62–29.53 in (600–750 mm)
Weight	11.68 lbs (5.3 kg)	12.13 lbs (5.5 kg)	8.38 lbs (3.8 kg)

Materials			
Toilet Lid	PP		
Toilet Seat	PPGF20		
Pan with Lid	PE		
Armrests	PE		
Splash Guard	PE		
Frame	Steel, powder coated	Steel, powder coated	Aluminum

3 Safety

3.1 Safety Information

WARNING!

Risk of Injury or Damage

- DO NOT use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as user manuals, service manuals or instruction sheets supplied with this product or optional equipment. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment.
- DO NOT use if defective.
- DO NOT make any unauthorized alterations or modifications to the product.
- Make sure that all attaching hardware, screws, nuts and/or bolts are tight at all times.
- Protect the product against exposure to heat.

WARNING!

Risk of Injury or Damage

- Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories.
- Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

4 Setup

4.1 Installing/Removing the Seat Lid and Pail

IMPORTANT

Risk of Product Damage

- Before removing/installing seat and lid, allow the seat and lid to reach room temperature if exposed to cold. This will help prevent the seat clamps from breaking when being removed or installed.
- DO NOT use excessive force when installing or removing the seat and lid.

Installing

WARNING!

Risk of Injury or Damage

- When installing the seat onto the frame, make sure that the seat is evenly spaced between the side frames to ensure proper stability.
- When installing the seat and lid, install the seat first and then the lid.

1. Align the grooves **A** on the underside of the seat **B** with the commode frame **C** (Figure 2, page 1).
2. Push down firmly until the seat snaps into place.
3. Align the grooves **D** on the underside of the lid **E** with the seat and the commode frame (Figure 3, page 1).
4. Push down firmly until the lid snaps in place.
5. Raise the seat and lid.
6. Put the pail **F** into place (Figure 4, page 1).
7. Lower the seat.

Removing

When removing the seat or lid, remove the pail, lid and then the seat.

1. Raise the seat **B** and lid **E**
2. Remove the pail **F** (Figure 4, page 1).
3. Lower the seat and lid.
4. Grasp the back corner of the lid and lift up firmly (Figure 3, page 1).
5. Grasp the back corner of the seat and lift up firmly (Figure 2, page 1).

4.2 Adjusting the Height of the Commode

WARNING!

- Ensure that the snap buttons fully protrude through the same height adjustment hole of each leg extension. This ensures that the leg extensions are securely locked in position and that an even height adjustment is achieved.

1. Depress the snap button **G** (Figure 5, page 1).
2. Slide the leg extension **H** up or down to the desired adjustment hole (Figure 6, page 1).
3. Ensure the snap button protrudes fully through the adjustment holes of the leg extension.

There will be an audible click when the snap button fully engages.

4. Repeat STEPS 1–3 for the remaining three legs.

Make sure that all four legs are adjusted to the same height.

5 Usage

5.1 Using the Commode

WARNING!

Risk of falling!

- The commode seat **MUST** be installed before sitting on the commode.
- Users with limited physical capabilities should be supervised or assisted when using the commode.
- Leg extensions with rubber tips **MUST** be in contact with the floor at all times.
- **DO NOT** stand on this product.
- **DO NOT** use as a climbing aid.
- **DO NOT** use the product in the shower.
- **DO NOT** sit on the lid.
- The commode does not have a backrest. **DO NOT** use the lid as a backrest. **DO NOT** use the upper rear tube (Model 9630 or 9650) as a backrest. The upper rear tube is designed only to hold the lid up.

6 Maintenance

6.1 Maintenance

WARNING!

- Make sure that all attaching hardware, screws, wing nuts and/or bolts are tight at all times. Otherwise, injury or damage may occur.
- Inspect rubber tips on the leg extensions for rips, tears, cracks or wear or if they are missing. Replace them **IMMEDIATELY** if any of these conditions exist.

The product is maintenance-free, provided the cleaning and safety instructions outlined above are adhered to.

Check product regularly for damage and to ensure that it is securely assembled.

6.2 Cleaning and disinfection

The product can be cleaned and disinfected using commercially available agents.

Disinfect the product by wiping down all generally accessible surfaces with disinfectant (after disassembling the product, if possible).

IMPORTANT

- All cleaning agents and disinfectants used must be effective, compatible with one another and must protect the materials they are used to clean. For further information on decontamination in Healthcare Environments, please refer to 'The National Institute for Clinical Excellence' guidelines on Infection Control www.nice.org.uk/CG002 and your local infection control policy.
- Clean by hand.
- Clean without scouring agents.
- **DO NOT** autoclave.

7 After Use

7.1 Disposal

When the product can no longer be used, the product, accessories and the packaging materials should be sorted correctly. Please be environmentally responsible and recycle this product through your recycling facility at its end of life.

The disposal and recycling of used devices and packaging must comply with the applicable legal regulations.

7.2 Reuse

WARNING!

Risk of injury

- Always have a dealer inspect the product for damage before transferring the product to a different user. If any damage is found, **DO NOT** use the product.

The product is suitable for repeated use. The number of times it can be used depends on how often and in which way the product is used. Before reuse, clean and disinfect the product thoroughly and check the safety functions, and reset if necessary.

8 Warranty

8.1 Warranty

We provide a manufacturer's warranty for the product in accordance with our General Terms and Conditions of Business. Guarantee claims can only be made through the relevant specialist dealer.

For warranty information, contact Invacare.

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© 2013 Invacare® Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted.
Aquatec Operations GmbH
Invacare Corporation