Invacare® HI90 Bula

EN	Anti-slip bath mat User Manual	1
DE	Rutschfeste Badematte Gebrauchsanweisung	2
FR	Tapis de bain antidérapant Manuel d'utilisation	3
NL	Antislipbadmat Gebruiksaanwijzing	4

EN

I General

I.1 Symbols in this user manual

In this user manual, warnings are marked with symbols. The title next to the warning symbol indicates the level of danger.

WARNING

Indicates a dangerous situation which could lead to death or serious injury if not avoided.

CAUTION

Indicates a dangerous situation which could lead to minor injuries if not avoided.

IMPORTANT

Indicates a dangerous situation which could lead to damage if not avoided.

Useful tips, recommendations and information to ensure efficient and smooth operation.

Manufacturer

I.2 Service life

The expected service life of this product is three years when used in accordance with the safety instructions and correct use, stated in this manual. The effective service life can vary according to frequency and intensity of use.

I.3 Intended use

The anti-slip bath mat is intended to be used inside bathtubs or suitable showers as a non-slip surface for a secure hold while bathing or showering. Any other use is prohibited.

The anti-slip bath mat can help to prevent persons with reduced mobility from falling in bathtubs and showers.

2 Safety

2.1 Safety information


WARNING!

- Check all parts for shipping damage. If damage is noted, do not use. Contact carrier/dealer for further instruction.
- If you have any doubts about the restrictions on use of the product, please contact your dealer.
- Always ensure that the product is suitable for your requirements; in case of doubt, check with a health professional.
- Do not leave or store the product outdoors.
- Take particular care when using the product on damp surfaces.

3 Usage

3.1 Using the anti-slip bath mat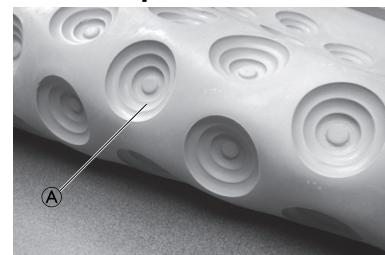

1. Place the anti-slip bath mat on the floor of the bathtub with suction cup openings **A** facing down.

The anti-slip bath mat can also be used in showers with minimum floor dimensions of 700 x 700 mm.

This manual **MUST** be given to the user of the product. **BEFORE** using this product, read this manual and save for future reference.

Yes, you can.®

4 Maintenance

4.1 Maintenance and periodic inspection

The product is maintenance-free, provided the cleaning and safety instructions outlined in this user manual are adhered to.

CAUTION!

– Before first use, clean the product thoroughly.

4.2 Cleaning and disinfection

IMPORTANT!

– Clean by hand.
– Do not use any aggressive (pH value under 4.5 or over 8.5) or abrasive cleaning agents.

The product can be cleaned and disinfected using commercially available agents.

1. Regularly remove the anti-slip bath mat out of the bathtub and clean it with a normal household cleaner.
2. Disinfect the product by wiping down all generally accessible surfaces with disinfectant.

5 Disposal

The disposal and recycling of used devices and packaging must comply with the applicable local legal regulations.

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Invacare Poirier SAS
Invacare GmbH,
Aquatec Operations GmbH